EN **Read the instructions carefully before use!**

GC Tooth Mousse

Water-based Sugar-free Tooth mousse with Recaldent™ CPP-ACP (Casein Phosphopeptide - Amorphous Calcium Phosphate)

Tooth mousse contents:

Pure water, glycerol, CPP-ACP, D-sorbitol, CMC-Na, Propylene glycol, Silicon dioxide, titanium dioxide, Xylitol, Phosphoric acid, Flavoring, Zinc oxide, Sodium saccharin, ethyl p hydroxybenzoate, Magnesium oxide, Guar gum, Propyl p-hydroxybenzoate, Butyl p hydroxybenzoate.

Indications for use:

1. To reduce sensitivity after ultrasonic, hand scaling or root planning.
2. Prior to or following teeth bleaching treatment.
3. For low salivary pH level during pregnancy.
4. For patients suffering from xerostomia due to medical treatments or as a side-effect of medical treatment.
5. To treat dental white-spot lesions.
6. During or after orthodontic treatment.
7. For high-risk caries patients or patients with a low salivary pH level.
8. For protection of patients with frequent consumption of high-acidity drinks (energy drinks, sodas).
9. For patients suffering from teeth sensitivity.

Contra-indications:

RECALDENT™ (CPP-ACP) is derived from milk casein. Not for use of patients with a proven or suspected milk protein allergy and/or with a sensitivity or allergy to benzoate preservatives.

Directions for use:

1. Prior to use, brush your teeth with standard toothbrush and toothpaste.

2. Extract a sufficient amount of mousse onto a clean and dry finger and apply to the teeth.
3. Wait for 3 minutes while avoiding spitting or swallowing saliva.
4. Use your tongue to apply the remnants of the mousse onto your teeth.
5. Spit the main left over mousses. The remaining residues will gradually dissolve. Avoid eating or drinking for at least 30 minutes. The more the application time lasts, the greater the activity of the mousse will be.

Tooth Mousse use frequency:

Use the GC Tooth Mousse twice a day – morning and evening after tooth brushing. During orthodontic treatment, apply the Tooth Mousse around the areas of the braces.

Storage

For optimal performance, store in a dry location at room temperature (8-25°C). Do not use the Tooth Mousse after the expiry date as stated on the package.

Caution

1. Replace cap immediately after use and remove any residual paste or moisture around the tube neck or inside the cap.
2. In case of contact with the skin, remove immediately by a wet cloth or gauze.
3. In case of contact with eyes, flush immediately with water and seek medical attention.
4. RECALDENT™ (CPP-ACP) is derived from milk casein. Do not use in case of proven or suspected milk protein allergy and/or with a sensitivity or allergy to benzoate preservatives.

RECALDENT™ is a registered trademark and is manufactured under license of Recaldent Pty. Ltd.

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